

# W THE WINSOR<sup>®</sup> TIE COMPANY

## Storage:

Sealed and placed in a dry, clean, non-corrosive gas place, clean, ventilated and odourless.  
Do not use it if it deteriorates during the validity period due to improper storage.



Adjust the nose clip  
to fit the face



Adjust ear hanger  
and face part



Put it in the bag after use

## User description:

The overall appearance of the mask before use should be free of damage of pollution, make sure the elasticity of the Strap is good and the nose clip is not damaged or broken.

Check whether the Exhaust valve disc is detached or damaged before use.

When you feel hard to breathe, dizziness or other discomfort, please leave the contaminated area immediately. Do not modify, clean, abuse or misuse this mask.

Patients with severe respiratory diseases, COPD, cardiovascular disease and pregnant women should use this mask with caution. Not recommended for children.